


Feces Collection Kit





MSK-FCK003 is a feces sampling device with transport medium which is used to collect, preserve and transport feces samples for testing and research purpose, it plays a crucial role in detecting various medical conditions such as gastrointestinal conditions, monitoring digestive health, and conducting research on gut microbiota. Feces collection kit includes a feces collection stick, a preservation tube with transport medium. The all-in-one system is perfect to preserve the feces samples for the real-time testing, customization is welcome.

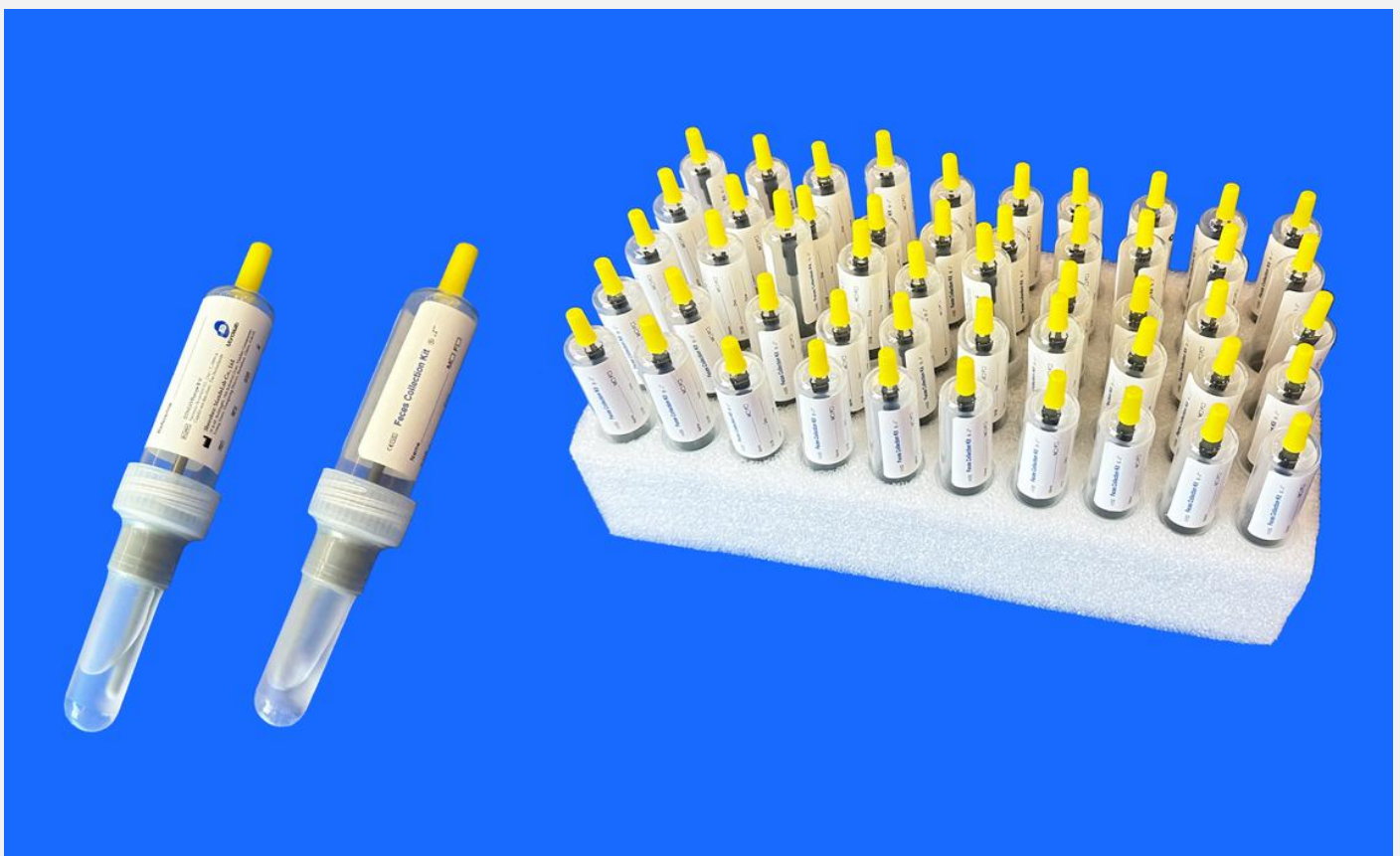

Collection Instructions (MSK-FCK003)

1. Take out the device, unscrew the cover and collect feces sample;
2. Put the stick with collected feces back into the cover and screw it;
3. Snap the stick inside of the tube to release the transport medium;
4. Shake the device to mix samples, and write down relevant information.

How to Use the Feces Collection Kit?

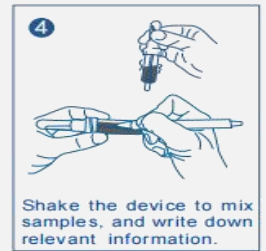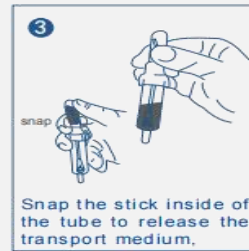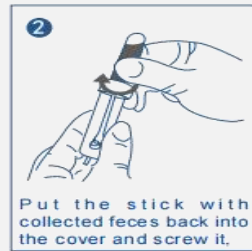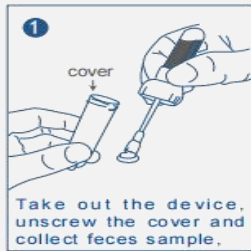

Product Features

- Physical performance is stable, with good sealing;
- Temperature tolerance ranges from -80 to 60°C;
- Non-invasive, easy collection, convenient transport;
- All-in-one Feces Collection Kit with Transport Medium.

Product Picture

